

MODEL-med

international pty. ltd.

ABN 23 492 360 470

***1296 Glenhuntly Rd, Carnegie 3163
Melbourne Australia***

Phone +61 418 506 308

Fax +61 3 9521 0517

www.modelmed.com.au

Gynaecological Trainer Care and Handling Instructions.

Congratulations on your purchase of the Model-med Gynaecological Trainer. If you follow one simple rule you shouldn't have any problems. That rule is:-

Treat the model as you would treat a person.

That is:-

1. Only use water-based lubricants.

Any standard water-based lubricant will work on the synthetic skin. Do not use oils, vaseline, petroleum jelly etc. Something like K-Y Jelly or its equivalent works best.

Never use the Trainer without using lubrication.

2. Be Gentle.

When inserting a speculum into the model, be gentle.

3. Clean the model immediately after use.

After use, you must clean the model with warm soapy water. Then pat dry with a towel, not a tissue. The synthetic flesh can feel slightly tacky, and sometimes a stubborn dirty mark can be picked up. Use a small amount of the special cleaning fluid with a clean white rag to remove the mark. If you have a very stubborn mark a bit of kerosene on a clean cloth can be used to remove it. Then re-wash as before with the soapy water.


4. Sprinkle with a little talcum powder.

When the models are dry you must dust them with a little talc to remove the tacky feel from the synthetic flesh, and to reduce the possibility of picking up dust and dirt. This also keeps the skin in good condition.


How to use the Model-med Gynaecological Trainer.

1

Having removed the model from the box you will find two "G" clamps. You can use these to fasten the model to the table.

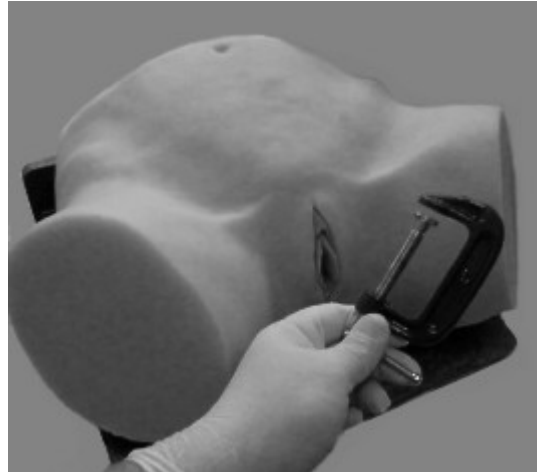

Choose the uterus you wish to use. Hold the clip as shown and gently insert the clip into the hole at the back of the uterus.

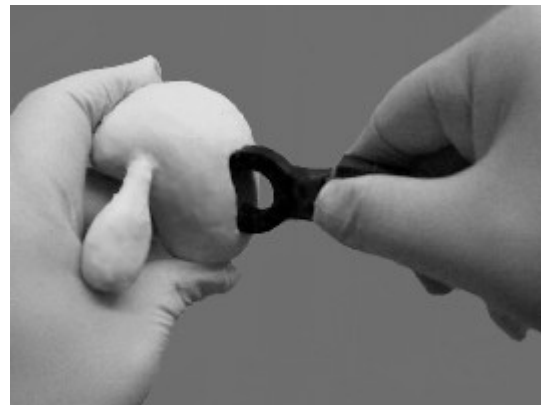

First lubricate the cervix with some water-based lubricant. This helps prevent sticking or dragging on the skirt. Thread the clip through the hole at the back of the "skirt" and position the cervix so it seats into the gap at the top of the birth canal.



Screw the clip onto the back of the model. It is now ready to use. You can exchange any of the uteri by repeating this procedure.

