

MODEL-med


international pty. ltd.

ABN 23 492 360 470

1296 Glenhuntly Rd, Carnegie 3163
Melbourne Australia

Phone +61 (0)418 506 308 www.modelmed.com.au

Sometimes the rubber in Model-med models will be dented or flattened if left for a long time in one place.

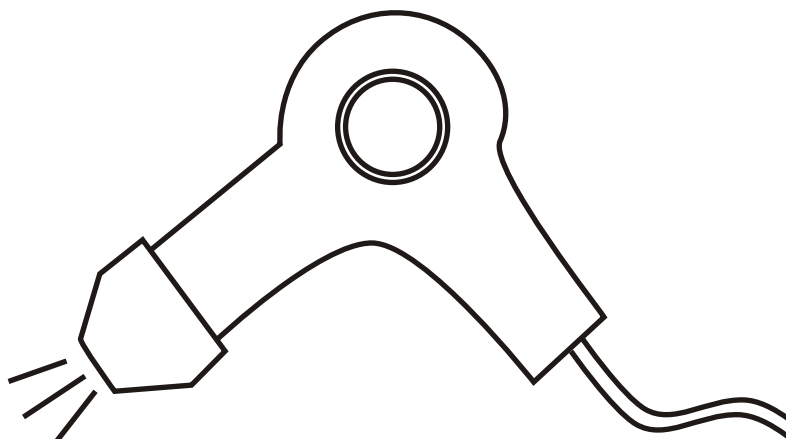

If you find that the rubber is deformed, gently warm the rubber with a hairdrier. But be careful not to overheat the rubber. A gentle heat is all that is needed.



This will bring the rubber back to its original shape.

MODEL-med

international pty. ltd.

ABN 23 492 360 470

1296 Glenhuntly Rd, Carnegie 3163
Melbourne Australia

Phone +61 (0)418 506 308

www.modelmed.com.au

"Charlie" Obstetric Trainer Care and Handling Instructions.

Congratulations on your purchase of the "Charlie" Obstetric Trainer. If you follow one simple rule you shouldn't have any problems. That rule is:-

Treat the model as you would treat a person.

That is:-

1. Only use water-based lubricants.

Any standard water-based lubricant will work on the synthetic skin. Do not use oils, vaseline, petroleum jelly etc. Something like K-Y Jelly or its equivalent works best.

Never use the Trainer without using lubrication.

2. Be Gentle.

When pulling a cervix over Charlie's head, and when inserting Charlie's head into the birth canal, be gentle.

3. Clean Charlie and his mum immediately after use.

After use, you can clean Charlie's head, the cervixes, and Charlie's mum with warm soapy water. Then pat dry with a towel, not a tissue. The synthetic flesh can feel slightly tacky, and sometimes a stubborn dirty mark can be picked up. Use a small amount of the special cleaning fluid with a clean white rag to remove the mark. Then re-wash as before with the soapy water. If you have a very stubborn mark a bit of kerosene on a clean cloth can be used to remove it. Then re-wash with soapy warm water, then pat dry.

4. Sprinkle with a little talcum powder.


When the models are dry you can dust them with a little talc to remove the tacky feel from the synthetic flesh, and to reduce the possibility of picking up dust and dirt.

5. Be careful with sharp objects, including fingernails.

You can damage the models if you use anything sharp on them, including long fingernails.

How to use the "Charlie" Obstetric Trainer.

Having removed Charlie's mum from the box you will find two "G" clamps. You can use these to fasten the model to the table.

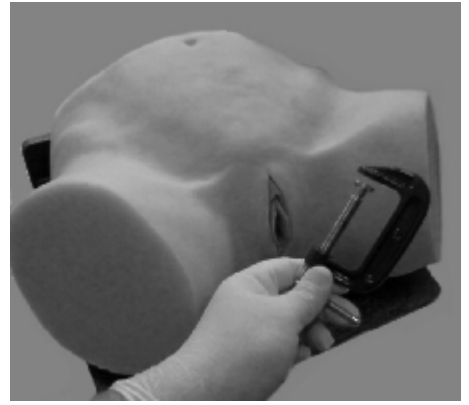


Choosing one of the cervixes, place it on Charlie's head like a cap. Simply by choosing where to place it on his head you can choose how the head appears through the cervix. Without using any lubricant gently begin stretching the cervix over Charlie's head until it is dilated by the desired amount. (If you use lubricant on his head the cervix will slip back instead of dilating.)







Now apply a liberal amount of water-based lubricant around the cervix and to the inside of the birth canal.



Place Charlie's head at the opening at the back of the model and gently push until it is fully inserted.

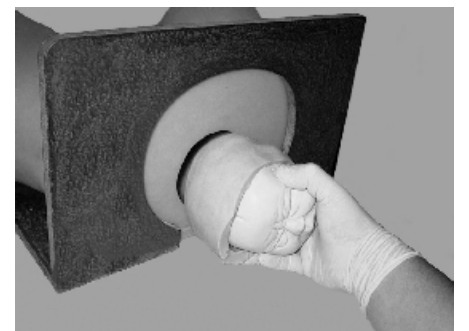

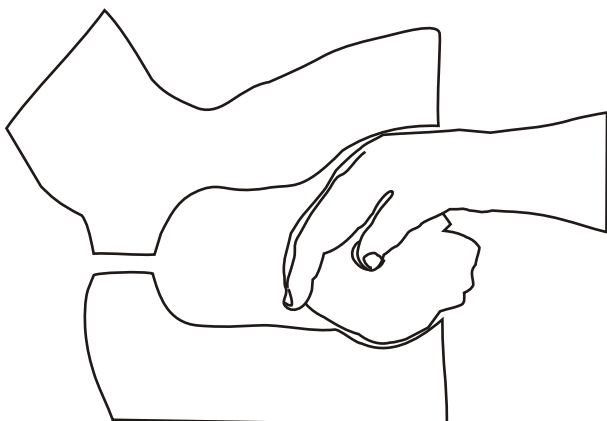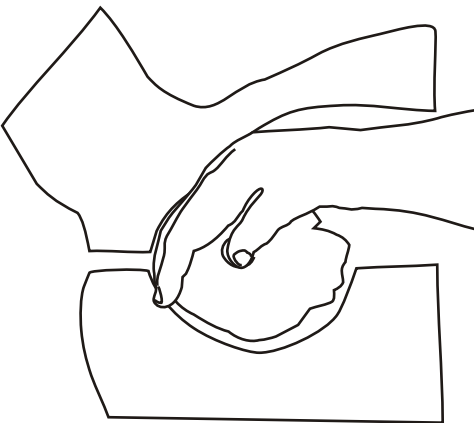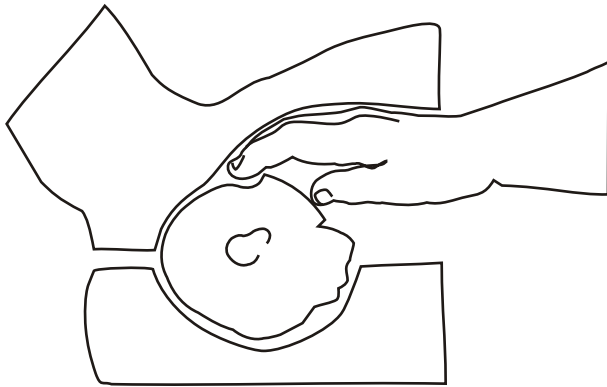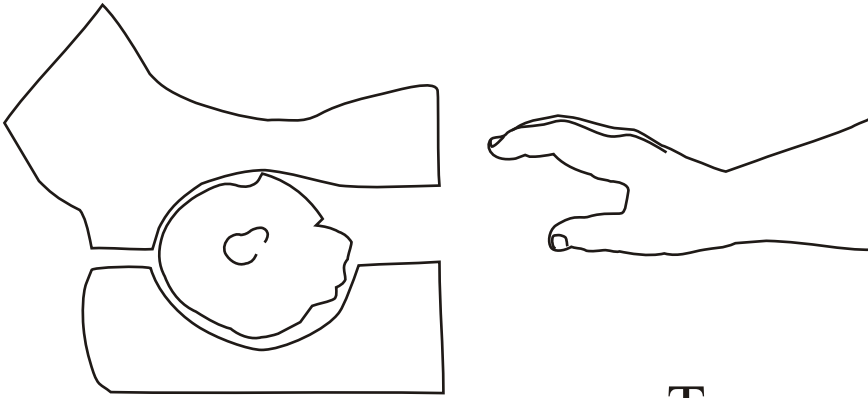

To remove the Charlie head, lubricate your hand and slide it inside the top of the birth canal at the rear of the model. Gently cup it over the front of the head and cervix and pull them firmly backwards and out.

MODEL-med
international pty. ltd.

ABN 23 492 360 470

Melbourne Australia

Phone +61 (0)418 506 308

Model-med Medical International Pty Ltd Warranty

Our products are made from a soft, yet tough, material. They will last if they are treated like a person. That is, gently.

Model-med International Pty Ltd offers a one year warranty to the original purchaser from the date of purchase on our models when they are handled, operated and maintained in accordance with the printed instructions accompanying them. This Limited Warranty shall be limited to the repair or replacement of parts that prove defective under normal use and service and which on examination shall indicate, to Model-med's satisfaction, they are defective. If Model-med confirms the defect and approves the claim, Model-med will elect to repair or replace, or have parts repaired or replaced without charge by a nominated representative of Model-med 's choosing. We may ask for photographic evidence of the damage to the model. The Warranty does not cover any costs of shipping. Shipping costs will be paid by the Customer.

This Limited Warranty does not cover any failures or operating difficulties due to accident, abuse, misuse, alteration, misapplication, vandalism, improper maintenance, or failure to perform normal and routine maintenance.

We will repair a model if it fails within one year of purchase providing it has not been mistreated or abused. We define mistreated or abused in, but not limited to, the following ways:-

If the model is cut, punctured or sliced by any object, including but not limited to forceps, knives, or any other implements.

Damage caused by fingernails is not covered by this warranty.

If any lubricant other than a water-based lubricant is used on the model. Only recognised water-based surgical lubricants should be used.

If any chemicals other than those described in the care and handling documents are used on the skin, or if the chemicals described in the care and handling documents are used in quantities greater than those recommended.

Hot objects should not be placed on or around the model. We define hot as being above 50 degrees celcius.

If you believe you have a claim under warranty you must first contact your local distributor for instructions on how to proceed with the claim.

Model-med International Pty Ltd shall not be liable under this or any implied warranty for incidental or consequential damages.