

# MODEL-med

international pty. ltd.


ABN 23 492 360 470

**1296 Glenhuntly Rd, Carnegie 3163**  
**Melbourne Australia**

Phone +61 (0)418 506 308 [www.modelmed.com.au](http://www.modelmed.com.au)



**Sometimes the rubber in Model-med models will be dented or flattened if left for a long time in one place.**

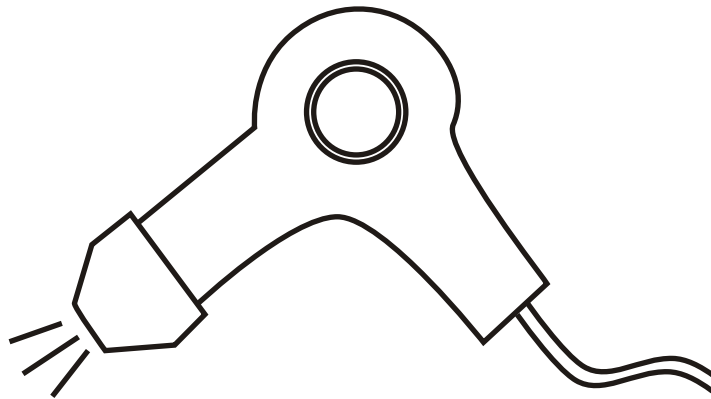

**If you find that the rubber is deformed, gently warm the rubber with a hairdrier. But be careful not to overheat the rubber. A gentle heat is all that is needed.**



**This will bring the rubber back to its original shape.**

**MODEL-med**  
international pty. ltd.

ABN 23 492 360 470

**1296 Glenhuntly Rd, Carnegie 3163**  
**Melbourne Australia**

Phone +61 (0)418 506 308 [www.modelmed.com.au](http://www.modelmed.com.au)

## **“Lucy and Lucy’s Mum” Instrumental Delivery Trainer Care and Handling Instructions**

Congratulations on your purchase of the “Lucy and Lucy’s Mum” Instrumental Delivery Trainer.

If you follow one simple rule you shouldn’t have any problems. That is:-

**Treat the models as you would treat a person**

That is:-

### ***1. Only use water based lubricants.***

Any standard water-based lubricant will work on the synthetic skin. Do not use oils, vaseline, petroleum jelly etc. Something like K-Y Jelly or its equivalent works best. Use the lubricant in VERY GENEROUS quantities.

**NEVER USE THE TRAINER WITHOUT USING LUBRICATION.**

**WHEN USING FORCEPS MAKE SURE THE BLADES ARE GENEROUSLY LUBRICATED EVERY TIME.**

***Please note that Ultra-sound gel does not work.***

### ***2. Be gentle.***

When picking up the models, and when inserting and pushing Lucy through the birth canal be firm without being rough.

### ***3. Clean Lucy and her Mum immediately after use.***

After use you can clean Lucy and her Mum with warm soapy water, but **do not immerse either in water**. Instead wipe them down with a warm soapy cloth. Then rinse with a clean wet cloth, and then pat dry with a clean towel, not a tissue. The synthetic flesh can feel slightly tacky, and sometimes a stubborn mark can be picked up. Use a small amount of the special cleaning fluid with a clean white cloth to remove the mark. Then re-wash as before. If you have a very stubborn mark then a small quantity of Kerosene on a clean cloth can be used to remove it.

Then re-wash as before, then pat dry.

### ***4. Sprinkle with a little talcum powder.***

When the models are dry you can dust them with a little talc to remove the tacky feel from the synthetic flesh, and to reduce the possibility of picking up dust and dirt.

## **How to use Lucy and her Mum**

Simply coat the Lucy head and the birth canal of Lucy's Mum liberally with a water-based lubricant such as K-Y Jelly. Now insert the Lucy head into the birth canal in the position you desire. You can now apply a vacuum cup or forceps to the head and practice instrumental delivery techniques.

As stated before, if using forceps please coat the blades generously with lubricant **EACH TIME** you use them.

**MODEL-med**  
international pty. ltd.

ABN 23 492 360 470

**1296 Glenhuntly Rd, Carnegie 3163**  
**Melbourne Australia**

Phone +61 (0)418 506 308 [www.modelmed.com.au](http://www.modelmed.com.au)

## **Model-med Medical International Pty Ltd Warranty**

**Our products are made from a soft, yet tough, material. They will last if they are treated like a person. That is, gently.**

**Model-med International Pty Ltd offers a one year warranty to the original purchaser from the date of purchase on our models when they are handled, operated and maintained in accordance with the printed instructions accompanying them. This Limited Warranty shall be limited to the repair or replacement of parts that prove defective under normal use and service and which on examination shall indicate, to Model-med's satisfaction, they are defective. If Model-med confirms the defect and approves the claim, Model-med will elect to repair or replace, or have parts repaired or replaced without charge by a nominated representative of Model-med 's choosing. We may ask for photographic evidence of the damage to the model. The Warranty does not cover any costs of shipping. Shipping costs will be paid by the Customer.**

**This Limited Warranty does not cover any failures or operating difficulties due to accident, abuse, misuse, alteration, misapplication, vandalism, improper maintenance, or failure to perform normal and routine maintenance.**

**We will repair a model if it fails within one year of purchase providing it has not been mistreated or abused. We define mistreated or abused in, but not limited to, the following ways:-**

**If the model is cut, punctured or sliced by any object, including but not limited to forceps, knives, or any other implements.**

**Damage caused by fingernails is not covered by this warranty.**

**If any lubricant other than a water-based lubricant is used on the model. Only recognised water-based surgical lubricants should be used.**

**If any chemicals other than those described in the care and handling documents are used on the skin, or if the chemicals described in the care and handling documents are used in quantities greater than those recommended.**

**Hot objects should not be placed on or around the model. We define hot as being above 50 degrees celcius.**

**If you believe you have a claim under warranty you must first contact your local distributor for instructions on how to proceed with the claim.**

**Model-med International Pty Ltd shall not be liable under this or any implied warranty for incidental or consequential damages.**